norditalia

e l e t t r o m e d i c a l i

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AEROSOL THERAPY SYSTEM FOR DOMESTIC USE

6

IMPORTANT WARNINGS: The use of any electrical appliance requires the respect of certain fundamental rules:

- Read all the instructions carefully before using the appliance;
- Respect the electrical specifications indicated on the appliance;
- The inhalation of any substance must be agreed upon with a doctor, who will indicate the most suitable doses and treatment methods;
- Electricity supply failure, an unexpected fault or any other unfavourable condition may cause the appliance not to work; we therefore recommend that you have an appliance or medicine (subject to agreement with your doctor) that may be used as an alternative;
- Only use the appliance for the use foreseen (aerosol inhalations) and as described in this manual;
- Do not let children or incapable persons use the appliance without supervision; in such cases, pay special attention during treatment;
- Do not use the appliance if the appliance housing or power supply cable are damaged;
- Do not leave packaging elements (plastic bags, cardboard boxes, etc.) within the reach of children or incapable persons as they represent a potential source of danger;
- If necessary, use adapters or extension leads that are compliant with the safety standards in force, paying attention not to exceed the rated current value and the maximum power value indicated on the adapter;
- Do not use the appliance with wet or damp hands or while taking a bath;
- Do not wet the appliance with water or other liquids; do not use the appliance if it has accidentally become wet;
- Keep the appliance and the power supply cable away from sources of heat;
- The appliance is fitted with common protection against the entry of liquids;
- The appliance is not suitable for use in the presence of inflammable substances;
- Do not use the appliance in rooms where sprays have just been used; air the room well before proceeding with the therapy;
- Do not block or obstruct the appliance air openings and do not place it on a soft surface such as a bed or settee;
- Do not insert any objects into the cooling openings;
- Do not use the appliance in cases where it makes unusual noises;
- Only use original accessories to avoid compromising the functionality of the appliance;
- We recommend personal use of the accessories;
- Do not pull the power supply cable or use the appliance to remove the plug from the power supply socket;
- Always remove the plug after use;
- Do not leave the appliance exposed to atmospheric agents; store it in the environmental conditions indicated.

RESPONSIBILITY

The manufacturer, his representative, or the importer shall only be considered responsible in terms of safety, performance and reliability if:

- the assembly technique, calibration, repairs or amendments have been performed by persons authorised by them;
- the electrical system complies with the standards in force;
- the operating instructions have been respected;

The manufacturer declines all responsibility for damage deriving from improper, erroneous and unreasonable use.

SAFETY FUSE

A safety fuse is housed within the appliance and it may be replaced if it blows.

As for all other possible repairs, the replacement operation must be performed by the manufacturer or by personnel authorised by the same.

ELECTROMAGNETIC COMPATIBILITY

The appliance is compliant with the electromagnetic compatibility standards currently in force and is suitable for use in all buildings, including those for residential / domestic use. The RF emissions from the appliance are very low and are not likely to cause any interference with nearby appliances.

In any case, we recommend that you avoid positioning it on top of or near other appliances. If it does appear to be interfering with your electrical appliances, move it or connect it to a different electrical socket. Radio appliances may affect the functioning of the appliance.

OPERATING INSTRUCTIONS

1. Position the appliance on a stable surface.
2. When using the appliance for the first time, clean the accessories as indicated in the "cleaning the accessories" section.
3. Connect the nebulizing ampoule (2) to the appliance, using the tube (1).
4. Connect one of the desired accessories to the nebulizing ampoule: mouthpiece (4), nosepiece (5) or mask (6), as indicated in the accessory connection diagram on page 2.
5. Insert the medication into the nebulizing ampoule, taking care not to accidentally spill any onto the appliance.
6. Insert the plug into the power supply socket after having checked that the power supply voltage corresponds to the rated voltage indicated on the label located under the appliance.

7. Start the appliance using the switch, bearing in mind that it may be tested for intermittent operation; The operating times are indicated in the table on page 16. The emission of vapour from the ampoule confirms correct functioning. When the appliance is new it is normal for the compressor and accessories to have a peculiar smell which leaves after few uses.

8. Inhale the nebulized medicine
9. Once the application has been completed, switch the appliance off and remove the plug from the power supply.
10. Clean the appliance and the accessories as indicated below.

**Important instructions about using the nebulizing ampoule.**

The plastic Neboplus ampoule (2) can be used in two different ways:

- with the plug (3) inserted, for slow nebulization (Fig. A);
- with the plug (3) open, for standard nebulization (Fig. B).

Fig. A

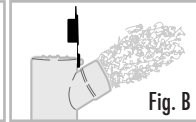
Fig. B

Attention: the nosepiece can only be used with the plug (3) inserted in the ampoule.

CLEANING THE APPLIANCE

All cleaning operations must be performed with the plug of the appliance removed from the power supply socket. To avoid anomalous appliance operation and the inhalation of undesired substances, clean the appliance and the accessories immediately after performing the treatment. The appliance and the accessory storage area must be cleaned using a cloth slightly moistened with alcohol. After cleaning the appliance, wait until the alcohol completely evaporates before re-use.

Do not use other liquids or substances. Do not use excessively wet cloths, if liquids come into contact with the electrical parts of the appliance, operating defects or irreparable damage may be caused.

CLEANING THE ACCESSORIES

After use, and if necessary, before use, wash the accessories (not including the tube) carefully with tepid water and dry them with a cloth. Then put the accessories into a clean storage area. When cleaning, ensure that all residue is removed and that the accessories are dried properly. Under no circumstances should you use substances that may be toxic if they come into contact with the skin or mucous, if ingested or inhaled. If it is necessary to disinfect or sterilize the accessories, adopt the methods approved by the regulations in force and in compliance with the resistance of the material.

Any residual humidity in the tube may be removed by leaving the aerosol on without the ampoule connected for a few minutes. If the tube is dirty it should be replaced.

The accessories (not including the tube) may be sterilized using specific products such as the MILTON® method: they should be rinsed to remove all traces of disinfectant, dried and stored in a dry and clean environment. The accessories (not including the tube) may be sterilized in boiling water (100°C) for at least 40' or in an autoclave (120°C) for at least 20'. Do not put weights on the accessories and do not exceed the temperatures indicated; use a sterile bag. PVC accessories will lose their transparency in water or vapour; this phenomenon will disappear after a few days in a dry environment and will not affect usability. All the accessories may be sterilized up to a maximum of 100 times, following which they should be replaced.

The frequency and interval between one sterilization session and another should be established according to the discretion of the patient and/or the clinic and according to the pathology. The accessories should normally be sterilized prior to each change of patient.

REPLACING THE FILTER

The appliance is fitted with a filter, which eliminates any impurities from the air sucked in by the compressor.

On a periodical basis, or if the appliance is no longer efficient, check the conditions of the filter, if it is very dirty it should be replaced. For checking or replacement, lift up the plug of the filter-holder compartment. If necessary, extract the filter using some tweezers. (See diagram on page 2 and table on page 16).

STANDARD ACCESSORIES

The standard accessories include one of the combinations indicated in the connection diagram (page 2):

- Nebulizing ampoule • Mask
- Mouthpiece • Nosepiece • Spare filter

All the accessories are compatible with direct skin contact.

ACCESSORY SPARE PARTS

To guarantee product performance, functionality and safety, only buy original Norditalia spare parts.

USEFUL TIPS

To preserve the efficiency of the appliance we recommend replacing the nebulizing ampoule every 12 months (Every 6 months for frequent use - 3 applications per day). Damaged accessories should be replaced.

WASTE DISPOSAL

Dispose of the appliance and its accessories in compliance with the regulations in force. Do not dispose of this appliance in the environment!

GUARANTEE CONDITIONS

This appliance is guaranteed for the period indicated in the table on page 16, except where otherwise indicated.

The guarantee is only valid if the device is returned to the stockist, together with its receipt as proof of the date of purchase.

The guarantee covers the appliance for any fabrication defects, on condition that the same has not been tampered with by the user or by persons other than the authorised maintenance service personnel.

The guarantee does not include:

- faults or defects caused by improper use, or failure to comply with the operating instructions;
- interventions for presumed defects, checks on request, ordinary maintenance and components subject to natural wear and tear;
- accidental damage such as shocks, falls, etc.

NB. For repairs or overhauls, return the appliance to the stockist in clean condition

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