| N° | ACCESSORI IN DOTAZIONE - CODICE        | ZUBEHÖR - CODE                                 | N° EQUIPO DE ACCESORIOS - CODIGO      | ACCESSOIRES FOURNIS - CODE            | ACCESSORIES SUPPLIED - CODE    | BIJGELIJDIGE ACCESSOIRES - CODE         |
|----|--|--|---------------------------------------|---------------------------------------|--------------------------------|---|
| 1  | tubo di connessione corto              | kurzer Verbindungsschlauch                     | tubo de conexión corto                | tube de connexion court               | short connection tube          | korte verbindingsslang                  |
| 2  | vaso di raccolta 250 ml                | sammelgefäß 250 ml                             | recipiente de recolección 250 ml      | vase de réception 250 ml              | 250 ml collection vessel       | verzamelpotje 250 ml                    |
| 3  | tubo di connessione lungo              | langer Verbindungsschlauch                     | tubo de conexión largo                | tube de connexion long                | long connection tube           | lange verbindingsslang                  |
| 4  | tubo di raccordo al "sondino"          | anschlußschlauch zur sonde                     | tubo de unión con la sonda            | tube de raccordo a la "petite sonde"  | connection tube to the "probe" | slangetje voor aansluiting aan de sonde |
| 5  | piccolo tappo riduttore di depressione | kleiner verschluß zur reduktion des tiefdrucks | taponcito reductor de depresión       | petit bouchon reducteur de dépression | small vacuum reducer plug      | klein dopje voor verminderend onderdruk |
| 7  | supporto del vaso di raccolta          | sammelgefäß-halterung                          | soporte del recipiente de recolección | support du vase de réception          | collection vessel support      | metaalband verzamelpotje                |
| 6  | ACCESSORI OPZIONALI                    | ZUBEHÖR AUF ANFRAGE                            | ACCESSORIOS OPTATIVOS                 | ACCESSOIRES EN OPTION                 | OPTIONAL ACCESSORIES           | EXTRA ACCESSOIRES                       |
|    | coppetta tiralatte                     | milchpumpe-schröpfkopf                         | copa sacaleche                        | ventouse tire-lait                    | breast pump cup                | schaaltje                               |

0900374008 Ed. 09/2007

2

ASPIRATORE AD USO DOMESTICO PER TRACHEOTOMIZZATI / TIRALATTE  
 ABSAUGPUMPE FÜR DEN HAUSGEBRAUCH, FÜR TRACHEOTOMIE-PATIENTEN BZW.  
 ALS MILCHPUMPE ZU VERWENDEN  
 ASPIRADORAS DOMESTICAS PARA TRACHEOTOMIZADOS / SACALECHE  
 ASPIRATEUR A USAGE DOMESTIQUE POUR TRACHEOTOMISE / TIRE-LAIT  
 ASPIRATOR FOR DOMESTIC USE FOR TRACHEOTOMISED PERSONS / BREAST PUMPS  
 HUISHOUELIJK ZUIGAPPARAAT VOOR PATIËNTEN MET TRACHEASNEDE /  
 BORSTPOMP

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## ASPIRATOR FOR DOMESTIC USE

This device can be used as a breast pump or as an aspirator for tracheotomised patients by separately purchasing the appropriate accessories depending on the use foreseen. It requires no lubrication, is robust, quiet and practical to use. The appliance is intended for one single patient and must be exclusively used in the home.

### IMPORTANT WARNINGS:

The use of any electrical appliance requires several fundamental rules to be observed:

- Read all the instructions carefully before using the appliance;
- Using the device must be agreed with your doctor, who will explain how to use it and the accessories to use; the device must anyway be used observing the prescriptions of this use manual;
- Electricity supply failure, an unexpected fault or any other unfavourable condition may cause the appliance not to work; we therefore recommend procuring an appliance that could be used as an alternative;
- Only use the appliance for the use foreseen (breast pumps or for tracheotomised patients) and as described in this manual;
- Do not let children or incapable persons use the appliance unsupervised; in these cases, pay special attention during treatment;
- If the appliance housing or power supply cable become damaged, do not use the appliance;
- Do not leave packaging elements (plastic bags, cardboard boxes, etc.) within the reach of children or incapable persons as they represent a potential source of danger;
- If necessary, use adapters or extension leads which conform to the safety standards in force, paying attention not to exceed the rated current value and the maximum power value indicated on the adapter or extension lead;
- Do not use the appliance with wet or damp hands or while taking a bath;
- Do not wet the appliance with water or other substances; do not use the appliance if it has accidentally become wet;
- Always use the collection vessel supplied as it has a safety valve which protects the appliance if the vessel becomes too full. We anyway advise against filling the vessel up to the point when the valve intervenes;
- Keep the appliance and the power supply cable away from sources of heat;
- The appliance has common protection against the ingress of liquids;
- The appliance is not suitable for use in the presence of an inflammable anaesthetic mixture with air, oxygen or nitrogen protoxide;
- Do not use the appliance in rooms where sprays have just been used; air the room well before use;
- Do not block or obstruct the appliance air openings and do not place it on a soft surface such as a bed or settee;
- Do not insert any objects into the cooling openings;
- Do not use the appliance if it makes a lot of noise;
- Only use original accessories to prevent compromising the functionality of the appliance;
- We recommend personal use of the accessories;
- Do not pull the power supply cable or use the appliance to remove the plug from the power supply socket;
- Always remove the plug after use;
- Do not leave the appliance exposed to atmospheric agents; keep it in the environmental conditions described below.

### RESPONSIBILITY

The manufacturer, fitter, installer or importer are only responsible in terms of safety, performance and reliability if:

- the assembly technique, calibration, repairs or modifications have been performed by persons authorized by them;
- the electrical system complies to the standards in force;
- the use instructions have been respected.

The producer declines all responsibility for damage deriving from improper, erroneous and unreasonable use.

### SAFETY FUSE

A safety fuse is housed in the appliance which can be replaced if it blows. As for all other possible repairs, the replacement operation must be performed by the producer company or by personnel authorized by the same. Before replacing the fuse, remove the plug from the power supply socket. Replace the fuse with one with the same electrical characteristics.

### ELECTROMAGNETIC COMPATIBILITY

The unit complies with the current standards regarding electromagnetic compatibility and is suited for use in all buildings, including homes and residential buildings.

The device produces very low radio frequency (RF) emissions and is not likely to cause interference with nearby equipment. It is, however, advisable not to set the unit on top of, or very close to, other electrical equipment. If it does generate any interference with your electrical equipment, move it or connect it to a different power outlet.

Mobile radio communication units could affect operation of the device.

### USE INSTRUCTIONS

See figure 1 to connect the tracheal probe (for applications during permanent tracheotomy).

See figure 2 to connect the breast pump cup.

- 1) Position the appliance on a stable surface.
- 2) When using the appliance for the first time, clean the accessories supplied as indicated in the point "CLEANING THE APPLIANCE AND THE ACCESSORIES". Visually check the integrity of collection vessel "2", of the entire length of the long connection tube "1" and of the short one "3", of small plug "5" and of all other accessories. Do not use the appliance if you find cracks, holes, encrustation or deformation of any kind.
- 3) (See Fig. 3) Check the efficiency of the safety valve in the plug of the collection vessel, checking that it slides vertically in the valve-support. The valve works through floatation and interrupts aspiration if the collection vessel becomes too full. In any event, prevent the level of the liquid from arriving as far as the valve (maximum recommended level - 240 ml.). If the safety valve intervenes, switch the appliance off and empty the collection vessel "2" as indicated below, in order to restore normal operation.
- 4) (See Fig. 3) To guarantee maximum device efficiency, check that the gaskets and the collection vessel plug components are positioned correctly and that the plug is closed correctly on the collection vessel. Deterioration or deformation of the components may influence the efficiency of the device causing vacuum leaks.
- 5) (See Fig. 4) Position the metal support "7" on the collection vessel "2" as indicated in the Figure. Next, hook the collection vessel onto the appliance using the support so that the vessel remains in an upright, stable position.
- 6) (See Fig. 1) Connect the "ASPIRAZIONE" of the appliance to point "A" of the collection vessel using the shortest tube "1", checking that no liquid has remained in the tube following cleaning. Next, connect the longest tube "3" to point "B" of the collection vessel.

Instructions for use with the tracheal probe (Figure 1):

- 7) Connect the small plastic tube "4" to the other end of the longest tube "3".
- 8) Connect the tracheal probe (purchased from a chemist) to the plastic tube "4".
- 9) To regulate or manually interrupt the aspiration force, place a finger on the hole "C" of the collection vessel "2".

Instructions for breast pump use (Figure 2):

- 7) Clean the cup (bought separately) as indicated in this manual. Connect the other end of the longest tube longest tube "3" to the breast pump cup "6" at point "E".
- 8) Insert the small perforated plug "5" supplied into hole "C" of the plug of the collection vessel "2".

Check that there are no residues inside the hole of the plug "5". If the hole becomes blocked this increases the aspiration force of the device which may cause injury.

- 9) To simulate the feeding of a baby act intermittently on the hole "D" of the breast pump cup "6" with a finger.

10) Insert the plug into the power socket, after checking that the power supply voltage corresponds to the rated voltage on the label under the appliance;

11) **Switch the appliance on, bearing in mind that it has been designed for intermittent operation of 60min. (On) / 60 min. (Off).**

If liquids are accidentally aspirated beyond the safety valve, into tube "1", switch the appliance off immediately and contact the stockist;

After use, switch the appliance off and remove the plug from the power socket. To empty the collection vessel, follow the instructions in point "CLEANING THE APPLIANCE AND THE ACCESSORIES".

12) Clean the appliance and the accessories supplied as in the successive paragraph.

### CLEANING THE APPLIANCE AND THE ACCESSORIES

All cleaning operations must be performed with the appliance plug removed from the power socket.

To prevent anomalous appliance operation, clean the appliance and the accessories immediately after use. For reasons of hygiene, avoid contact with fluids contained in the collection vessel and perform the cleaning operations using protective gloves (only for applications with permanent tracheotomy).

Cleaning the accessories supplied (See Figs. 1 and 4)

After each occasion of use, dispose of the liquid recovered in the collection vessel "2" - Do not use it under any circumstances.

Proceed as follows to empty the collection vessel "2":

- Switch off the appliance
- Remove the power supply plug from the current socket
- Disconnect the short tube "3" and the long tube "1" from the collection vessel plug
- Slide the collection vessel "2" out of its metal support "7"
- Unscrew the plug and pour the liquid into a suitable container for disposal. Do not perform any operation close to the appliance in order to avoid wetting or contaminating it
- Wash all the components as indicated in the manual.

Wash all the accessories with luke warm water (max. 40°C) and then sterilise them with a cold chemical disinfectant commonly found on sale (e.g. Milton® method, normally used for feeding bottles and teats) using it as indicated by the instructions for the disinfectant. Next, leave the accessories to dry on a clean cloth (do not use cotton or cloths that leave residues).

**Warning:** After performing the cleaning operations it is necessary to check that connection tube "1" (see Fig. 1-2) is completely dry before using again. For use as a breast pump check that the small perforated plug "5" (see Fig. 2) is free from residues inside and that the passage hole is free.

Cleaning the plug of the collection vessel (See Fig. 3)

To permit all the components of the collection vessel plug (See Fig. 3) to be cleaned well, and to guarantee the efficiency of the safety valve, we recommend only removing the components marked with an asterisk (\*) from the plug. To ensure efficiency of the valve (See Fig. 3) check that the visible part of the valve (black) gasket is free from encrustation or residues. If this is not the case, remove them without damaging the valve gasket.

Cleaning the breast pump cup (See Fig. 2)

To avoid contaminating the milk collected in the breast pump cup, we recommend sterilising it before use. For correct cleaning, use a cold chemical disinfectant commonly found on sale (we recommend the Milton® method), using it as foreseen by the producer's instructions.

Cleaning the appliance and the accessory space.

The appliance and accessory space must be cleaned using a cloth slightly dampened with alcohol. After cleaning the appliance, wait until the alcohol has completely evaporated before using it again. Do not use other liquids or substances. Do not use cloths which are excessively wet as any liquids possibly penetrating the appliance may cause operating defects or irreparable damage.

### DISPOSE OF THE APPLIANCE

Should it be necessary to dispose of the appliance or its accessories, contact specialised disposal services. **Dispose of correctly!**

### GUARANTEE CONDITIONS

This appliance is guaranteed for 2 years from the date of sale, as proved by the sales receipt which the purchaser must keep. The guarantee covers the appliance for any fabrication defects, on condition that the same has not been tampered with by the user or by persons foreign to the authorised service. The guarantee does not include:

- faults or defects caused by improper use or installation errors;
- interventions for presumed defects, checks, ordinary maintenance and components subject to natural wear;
- accidental damage such as shocks, falls, etc.
- the accessories and parts subject to normal wear.

The guarantee is only valid if the device is returned to the stockist, together with its receipt.

**NB. For repairs or overhauls return the appliance to the stockist in a clean condition.**

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